Specifications

Size	3650 Series7.7" W x 4.1" H x 13" D (19.5 cm x 10 cm x 33 cm)
	4650 Series7" W x 3.8" H x 13" D (18 cm x 10 cm x 33 cm)
	5650 Series10.1" W x 10.5" H x 6.5" D (26 cm x 27 cm x 17 cm)
Weight	3650 Series5.8 lbs. (2.6 kg)
	4650 Series5.7 lbs. (2.6 kg)
	5650 Series7.1 lbs. (3.2 kg)
Electrical	
Requirements	3650D2.3 amps, 115 VAC 60 Hz
	3650I1.4 amps, 230 VAC 50 Hz
	3650P1.4 amps, 230 VAC 60 Hz
	4650D & 465022.5 amps, 115 VAC 60Hz
	4650I1.4 amps, 230 VAC 50 Hz
	5650D & 5650H1.3 amps, 115VAC 60Hz
	5650I0.8 amps, 230VAC 50Hz
	5650P0.8 amps, 230VAC 50/60Hz

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Tamaño	3650 Serie19.5 cm x 10 cm x 33 cm (7,7" W x 4,1" H x 13" D)
	4650 Serie18 cm x 10 cm x 33 cm (7" W x 3,8" H x 13" D)
	5650 Serie26 cm x 27 cm x 17 cm (10,1" W x 10,5" H x 6,5" D)
Peso	3650 Serie2,6 kg (5,8 lbs.)
	4650 Serie2,6 kg (5,7 lbs.)
	5650 Serie3,2 kg (7,1 lbs.)
Nivel de sonido	3650 Serie58 dBA
	4650 Serie53 dBA
	5650 Serie51 dBA
Requerimientos	
Eléctricos	3650D2.3 amps, 115 VCA 60 Hz
	3650I1.4 amps, 230 VCA 50 Hz
	3650P1.4 amps, 230 VCA 60 Hz
	4650D y 465022.5 amps, 115 VCA 60Hz
	4650I1.4 amps, 230 VCA 50 Hz
	5650D y 5650H1.3 amps, 115VCA 60Hz
	5650I0.8 amps, 230VCA 50 Hz
	5650P0.8 amps, 230VCA 50/60 Hz

Sound Level	3650 Series58 dBA
	4650 Series53 dBA
	5650 Series51 dBA
Power Consumption	3650D140 watts maximum
	3650I & 3650P160 watts maximum
	4650D & 46502140 watts maximum
	4650I160 watts maximum
	5650D & 5650H90 watts maximum
	5650I & 5650P100 watts maximum

Max. Compressor

Pressure	3650 & 4650 series35 psig or greater*
	5650 series30 psig or greater*

Compressor Free Air Flow (3650 & 4650)8 lpm or greater* (5650)9 lpm or greater*

Nebulizer Operating Pressure all models11-16 psig

Nebulizer Liter Flow all models5-7 lpm

Nebulizer Capacity all models6 ml (cc)

Nebulizer Nebulization Rate all models0.15 ml/min or greater

MMAD all modelsLess than 5 microns

*Conditions may vary based on altitude above sea level, barometric pressure changes.

Consumo de Potencia	3650D140 watts maximo
	3650I y 3650P160 watts maximo
	4650D y 46502140 watts maximo
	4650I160 watts maximo
	5650D y 5650H90 watts maximo
	5650I y 5650P100 watts maximo

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Five-Year Limited Warranty on Compressor Parts

All DeVilbiss Compressors are warranted to be free from defective workmanship and material for a period of five years from the date of purchase. Any defective part(s) will be repaired or replaced at Sunrise's option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. If repair is necessary, contact your authorized Sunrise provider or Sunrise Service Department at 800-333-4000 (814-443-4881) for instructions. **NOTE**— This warranty does not cover providing a loaner compressor, compensating for costs incurred in compressor rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

Manufacturer's Note

Thank you for choosing a DeVilbiss compressor/nebulizer. We want you to be a satisfied customer. If you have any questions or comments, please send them to our address below.

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Printed in the U.S.A.
on recycled paper.

©Sunrise Medical 9.00
A-650 Rev. B

DEVILBISS

DeVilbiss® Compressor/Nebulizer Instruction Guide

Pulmo-Aide® Models 5650D, 5650I, 5650H & 5650P

PulmoMate® Models 4650D, 46502 & 4650I

Pulmo-Aide® LT Models 3650D, 3650I & 3650P

CAUTION—Federal (U.S.A.) law restricts this device to sale by, or on the order of a physician.

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Modelos 3650D, 4650D,
46502, 5650D & 5650H
Calificado U.L.

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

DANGER —Urgent safety information for hazards that will cause serious injury or death.

WARNING— Important safety information for hazards that might cause serious injury.

CAUTION — Information for preventing damage to the product.

NOTE —Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

DANGER —To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not use while bathing.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place in or drop into water or other liquid.
- Do not reach for a product that has fallen into water. Unplug immediately.

WARNING—To reduce the risk of burns, electrocution, fire or injury to persons:

- A product should never be left unattended when plugged in.
- Close supervision is necessary when this product is used by, on, or near children or invalids.
- Use this product only for its intended use as described in this guide. Do not use attachments not recommended by the manufacturer.
- Never operate this product if:
 - It has a damaged power cord or plug.
 - It is not working properly.
 - It has been dropped or damaged.
 - It has been dropped into water.
Return the product to an authorized Sunrise service center for examination and repair.
- Keep the power cord away from heated surfaces.
- Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
- Never use while drowsy or asleep.
- Never drop or insert any object into any opening or hose.
- Do not use outdoors. This product is for household use only.
- Do not use in an oxygen enriched environment.
- Connect this product (for grounded models) to a properly-grounded outlet only. See *Grounding Instructions*.

NOTE —This unit employs a thermal limiter that will shut off the entire product should the motor overheat.

5650—The unit will restart when the motor cools to an acceptable level.

4650/3650—Contact your qualified Sunrise provider for a replacement.

If repair or replacement of the cord or plug is necessary, contact your qualified Sunrise provider.

Model 5650H & 4650D Grounding Instructions

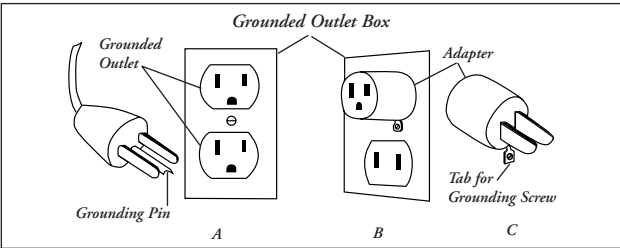
This product should be grounded. In the event of an electrical short circuit, grounding reduces the risk of electric shock by providing an escape wire for the electric current. This product is equipped with a cord having a grounding wire with a grounding plug. The plug must be plugged into an outlet that is properly installed and grounded.

DANGER— Improper use of the grounding plug can result in the risk of electric shock.

If repair or replacement of the cord or plug is necessary, contact your qualified Sunrise provider.

- This unit is for use on a nominal 115V circuit and has a grounding plug. A temporary adapter, which looks like the adapter illustrated in **Figures B and C**, may be used to connect this plug to a 2-pole receptacle (as shown in **Figure B**) if a properly grounded outlet is not available. The temporary adapter should be used only until a qualified electrician can install a properly grounded outlet (**Figure A**). The green colored rigid ear, lug, and the like extending from the adapter must be connected to a permanent ground such as a properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by the metal screw.
- If it is necessary to use an extension cord, use only a 3-wire extension cord that has a three-blade grounding plug, and a 3-slot receptacle that will accept the plug on the unit. If power cord is damaged do not use unit, and contact a qualified Sunrise Medical provider.

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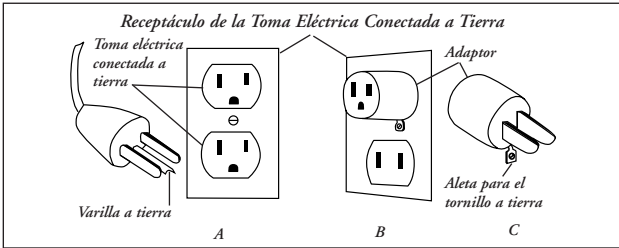
GUARDE ESTAS INSTRUCCIONES.

Introduction

Your doctor has prescribed a liquid medication to treat your respiratory condition. To best utilize this liquid medication, he has prescribed a DeVilbiss brand compressor/nebulizer. Your DeVilbiss compressor/nebulizer works to convert the medication to a high-quality mist of fine particles that penetrates deep into the lungs.

Be sure that you read and understand the information in this instruction guide. By following these simple instructions and the advice of your physician, your compressor will become an effective addition to your therapeutic routine.

Statement of Intended Use

The DeVilbiss compressor/nebulizer is an AC powered air compressor that provides a source of compressed air for home health care use. The product is used in conjunction with a jet (pneumatic) nebulizer to convert liquid medication into an aerosol form with particles smaller than 5 microns diameter for inhalation into the patient's respiratory tract for the treatment of respiratory disorders.

The target population for this device consists of both adult and pediatric patients suffering from, but not limited to asthma, cystic fibrosis, and chronic obstructive pulmonary disease. Further, additional applications for aerosolized medication are constantly under investigation and this device may be deemed suitable for such applications as prescribed.

The intended environment for use of the product is in the patient's home on the order of a physician.

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Important Parts of Your DeVilbiss Compressor

(Figures. A, B & C)

- Storage compartment
- Storage compartment door with operating instructions
- Power switch
- Filter cap
- Air-outlet connector
- Carrying handle
- Power cord storage
- Nebulizer holder

NOTE— Models 4650D & 5650H are equipped with a three-prong plug and must be grounded. All other models are equipped with a two-prong plug.

Important Parts of Your Disposable Nebulizer

- Mouthpiece
- Nebulizer t-piece
- Nebulizer cap
- Nebulizer chamber
- Nebulizer air-inlet connector
- Nebulizer baffle
- Tubing

Nebulizer parts are not sold separately. Additional nebulizers (part #4650D-620) are available through your Sunrise provider.

NOTE— Nebulizer received may not be the one shown.

Important Parts of Your Reusable Nebulizer

- Mouthpiece
- Cap
- Baffle
- Medicine Cup
- Interrupter Adapter (if applicable)

Nebulizer parts are not sold separately. Additional nebulizers are available through your Sunrise provider.

NOTE— Nebulizer received may not be the one shown.

How to Operate Your Compressor:

NOTE— Before initial operation, your nebulizer should be cleaned following cleaning instructions, or as recommended by your physician or Sunrise provider.

- Place compressor on a level, sturdy surface so that the controls can be easily reached when seated.
- Open door to storage compartment **Figure 1**.
- Make sure the power switch is in the "Off" position **Figure 2**. Unwrap power cord and plug power cord into an appropriate wall outlet (refer to *Specifications*). Plug models 5650H & 4650D into a grounded wall outlet (refer to *Grounding Instructions* on reverse).

DANGER— DeVilbiss compressor/nebulizer must be operated on the specified power source to avoid risk of electric shock and damage to the compressor.

- Wash hands.
- Connect one end of the nebulizer tubing to the air-outlet connector **Figure 3**.

NOTE— During high-humidity weather conditions, condensation (water build up) can occur in the internal hose of the compressor. It is recommended that the compressor be turned on and allowed to run for two (2) minutes before the tubing is attached to the air outlet connector.

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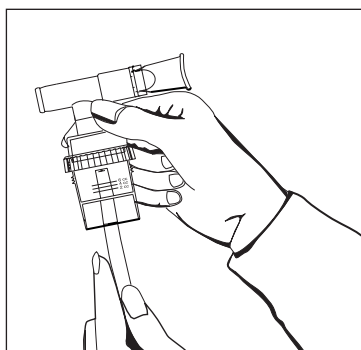
Figure 6/Figura 6

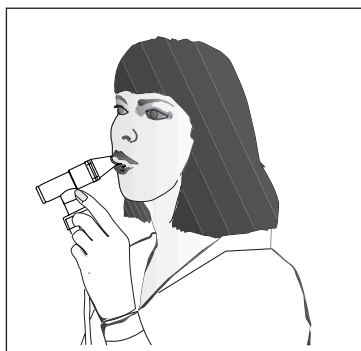
Figure 7/Figura 7

- Assemble clean nebulizer parts by placing baffle down in nebulizer chamber. Holding chamber stationary, screw on nebulizer cap. Add prescribed medication through the opening on cap using a medicine dropper or pre-measured dose container **Figure 4**.
- Assemble mouthpiece and t-piece (if applicable) and insert into the top of the nebulizer cap **Figure 5**. If using an aerosol mask, insert the bottom part of the mask directly into the top of the nebulizer cap.
- Attach tubing to nebulizer air-inlet connector **Figure 6**.
- Press the power switch "On" to start the compressor.
- Begin treatment by placing the mouthpiece between teeth. With mouth closed, inhale deeply and slowly through mouth as aerosol begins to flow then exhale slowly through the mouthpiece **Figure 7**. If treatment needs to be interrupted, simply press power switch "Off".

NOTE— Some physicians recommend a "clearing breath" after every five to seven treatment breaths. Remove mouthpiece from mouth and hold breath for at least five seconds (ten is better), then exhale slowly.

- If an aerosol mask is used, place the mask over mouth and nose **Figure 8**. As aerosol begins to flow, inhale deeply and slowly through mouth, then exhale slowly.

Nebulizer Cleaning

All parts of the nebulizer, except tubing, should be cleaned according to the following instructions. Your physician and/or Sunrise provider may specify a certain cleaning procedure. If so, follow their recommendations.

WARNING— To prevent possible risk of infection from contaminated medication, cleaning of the nebulizer is recommended after each aerosol treatment.

After Every Use:

- With power switch in the "Off" position, unplug power cord from wall outlet.
- Disconnect tubing from the air-inlet connector and set aside.
- Disassemble mouthpiece or mask from cap. Open chamber by turning cap counterclockwise and remove baffle.
- Wash all items, except tubing, in a hot water/dishwashing detergent solution. Rinse under hot tap water for 30 seconds to remove detergent residue. Allow to air dry.

Daily:

- Using a clean container or bowl, soak items in three parts hot water to one part white vinegar for 30 minutes **Figure 9** OR use a medical bacterial-germicidal disinfectant available through your provider. For reusable nebulizer only, clean daily in dishwasher using top shelf. Be sure to follow manufacturer's instructions carefully.
- With clean hands, remove items from disinfectant solution, rinse under hot tap water and air dry on a clean paper towel. Store in a zip-lock bag.

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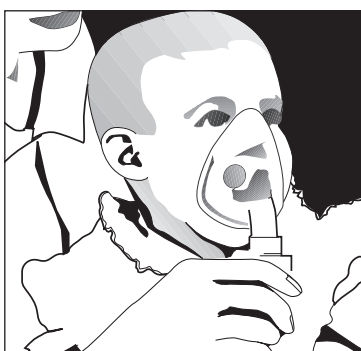
Figure 8/Figura 8

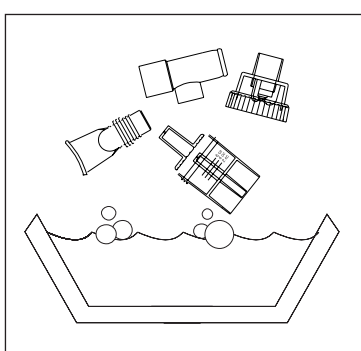
Figure 9/Figura 9

NOTE— Do not towel dry nebulizer parts; this could cause contamination.

CAUTION— The DeVilbiss Reusable Nebulizer is dishwasher safe, but do not place any disposable nebulizer parts in an automatic dishwasher; doing so could cause damage.

WARNING— To prevent possible risk of infection from contaminated cleaning solutions, always prepare fresh solution for each cleaning cycle and discard solution after each use.

- Keep the outer surface of the tubing dust-free by wiping regularly. Disposable nebulizer tubing does not have to be washed because only filtered air passes through it.

NOTE— The DeVilbiss Disposable Nebulizer will last at least 15 days and possibly longer, depending on usage. Proper cleaning will also help extend the life of the nebulizer. Because it is disposable, we recommend that an extra nebulizer be kept on hand at all times. Sunrise also manufactures the DeVilbiss Reusable Nebulizer that is dishwasher safe and may be cleaned and reused for up to one year.

Compressor Cleaning

- With power switch in the "Off" position, unplug power cord from wall outlet.
- Wipe outside of the compressor cabinet with a clean damp cloth every few days to keep dust-free.

CAUTION— Do not submerge in water; doing so will result in compressor damage.

Filter Change

- Filter should be changed every 6 months or sooner if filter turns completely gray in color.
- Remove filter cap by grasping it firmly and pulling out of the unit (**Figures A, B or C**).
- Remove dirty filter with fingers and discard.
- Replace with a new DeVilbiss filter (part #5650D-602). Additional filters should be purchased from your Sunrise provider.
- Push filter cap with new filter into position.

CAUTION— Reusing filter or substituting any other material such as cotton for a DeVilbiss air-inlet filter will result in compressor damage.

NOTE— An extra filter is packaged with the 4650 & 5650 series units.

Maintenance

All maintenance must be performed by a qualified Sunrise provider or authorized service center.

DANGER— Electric shock hazard. Do not remove compressor cabinet. All disassembly and maintenance must be done by a qualified Sunrise provider.

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