IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

DANGER—Urgent safety information for hazards that will cause serious injury or death.

WARNING—Important safety information for hazards that might cause serious injury.

CAUTION—Information for preventing damage to the product.

NOTE—Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

DANGER—To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not use while bathing.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place in or drop into water or other liquid.
- Do not reach for a product that has fallen into water. Unplug immediately.

WARNING—To reduce the risk of burns, electrocution, fire or injury to persons:

- A product should never be left unattended when plugged in.
- Close supervision is necessary when this product is used by, on, or near children or invalids.
- Use this product only for its intended use as described in this guide. Do not use attachments not recommended by the manufacturer.
- Never operate this product if:
 - It has a damaged power cord or plug.
 - It is not working properly.
 - It has been dropped or damaged.
 - It has been dropped into water.


Return the product to an authorized Sunrise

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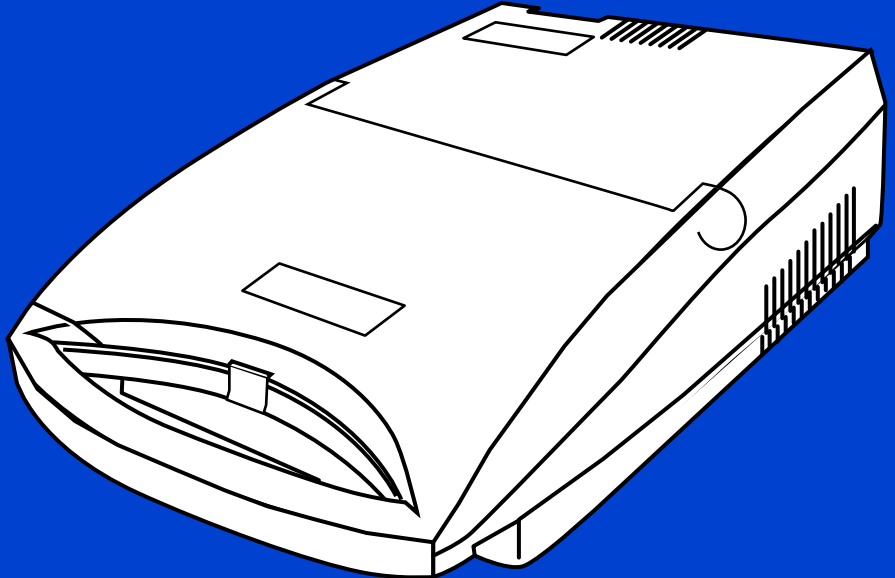
CAUTION —Federal (U.S.A.) law restricts this device to sale by, or on the order of a physician.
--

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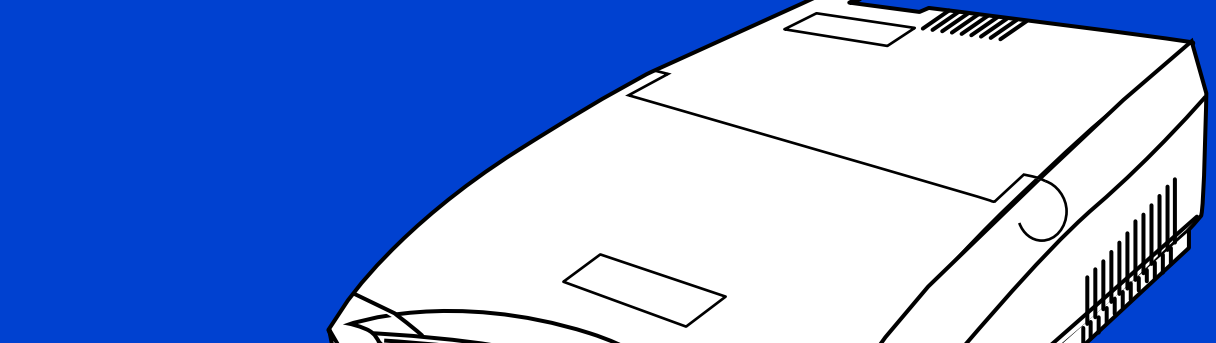service center for examination and repair.

- Keep the power cord away from heated surfaces.
- Never block the air openings of the product or place it on a soft surface, such as a bed or couch.

where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.

- Never use while drowsy or asleep.
- Never drop or insert any object into any opening or hose.

Do not use outdoors. This product is for household use only.

Do not use in an oxygen enriched environ-

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SAVE THESE INSTRUCTIONS

If repair or replacement of the cord or plug is necessary, contact your qualified Sunrise provider.

NOTE—This unit employs a thermal limiter that will shut off the entire product should the motor overheat.

WARNING—Information for preventing damage to the product.

CAUTION—Information for preventing damage to the product.

NOTE—Information to which you should pay special attention.

DANGER—Urgent safety information for hazards that will cause serious injury or death.

WARNING—Important safety information for hazards that might cause serious injury.

CAUTION—Information for preventing damage to the product.

NOTE—Information to which you should pay special attention.

DANGER—To reduce the risk of electrocution:

1. Always unplug this product immediately after using.

2. Do not use while bathing.

3. Do not place or store product where it can fall or be pulled into a tub or sink.

4. Do not place in or drop into water or other liquid.

5. Do not reach for a product that has fallen into water. Unplug immediately.

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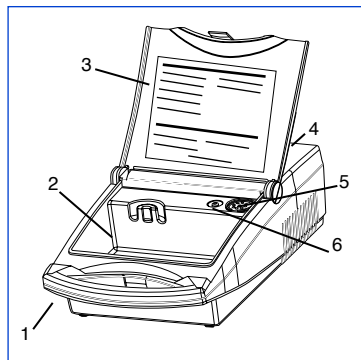
Important Parts of Your DeVilbiss Compressor

1. Carrying handle
2. Storage compartment
3. Storage compartment door with operating instructions
4. Power switch
5. Filter cap
6. Air outlet connector

NOTE—The Model 3650D is equipped with a two-prong plug.

Important Parts of Your Nebulizer

1. Mouthpiece
2. Nebulizer t-piece
3. Nebulizer cap
4. Nebulizer chamber
5. Nebulizer air-inlet connector
6. Nebulizer baffle
7. Tubing

Nebulizer parts are not sold separately. Additional nebulizers (part #4650D-620) are available through your Sunrise provider.

How to Operate Your Compressor:

1. Place compressor on a level, sturdy surface so that the controls can be easily reached when seated.

NOTE—Before initial operation, your nebulizer should be cleaned following cleaning instructions 4-6, or as recommended by your physician or Sunrise provider.

dy surface so that the controls can be easily reached when seated.

DANGER—DeVilbiss compressor/nebulizer must be operated on the specified power source to avoid risk of electric shock and damage to the compressor.

2. Lift door and swing up to open storage compartment **Figure 1**.
3. Make sure the power switch is in the "Off" position **Figure 2**. Unwrap power cord from storage area and plug power cord into a 115 VAC/60 Hz wall outlet (230 VAC/50 Hz for Model 3650I).
4. Wash hands.
5. Connect one end of the nebulizer tubing into the air outlet connector **Figure 3**.
6. Assemble clean nebulizer parts

NOTE—During high-humidity weather conditions, condensation (water build up) can occur in the internal hose of the compressor. It is recommended that the compressor be turned on and allowed to run for two (2) minutes before the tubing is attached to the air outlet connector.

by placing baffle down in nebulizer chamber. Holding chamber stationary, screw on nebulizer cap. Add prescribed medication through the opening on cap using a medicine dropper or pre-measured dose container **Figure 4**.

7. Assemble mouthpiece and t-piece and insert into the top of the nebulizer cap **Figure 5**. If using an aerosol mask, insert the bottom part of the mask directly into the top of the nebulizer cap.
8. Attach tubing to nebulizer air-

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Figure 8/Figura 8

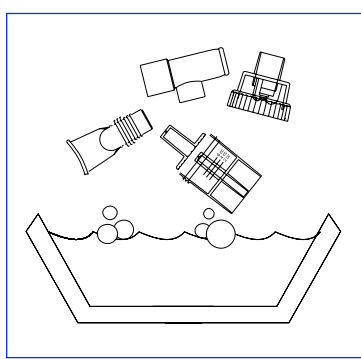
Figure 9/Figura 9

9. Press the power switch "On" to start the compressor.
10. Begin treatment by placing

NOTE—Cold temperatures can cause delayed motor start-up. Should this occur, place the compressor in a warm environment, such as a heated room. Allow the compressor to reach room temperature and press the power switch "On" again.

the mouthpiece between teeth. With mouth closed, inhale deeply and slowly through mouth as aerosol begins to flow then exhale slowly through the mouthpiece **Figure 7**. If treatment needs to be interrupted, simply press power switch "Off".

NOTE—Some physicians recommend a "clearing breath" after every five to seven treatment breaths. Remove mouthpiece from mouth and hold breath for at least five seconds (ten is better), then exhale slowly.

Figure 8. As aerosol begins to flow, inhale deeply and slowly through mouth, then exhale slowly.

Nebulizer Cleaning

All parts of the disposable nebulizer, except tubing, should be cleaned according to the following instructions after every use. Your physician and/or Sunrise provider may specify a certain cleaning procedure. If so, follow their recommendations.

1. With power switch in the "Off" position, unplug power cord from wall outlet.

WARNING—To prevent possible risk of infection from contaminated medication, cleaning of the nebulizer is recommended after each aerosol treatment.

2. Disconnect tubing from the air-inlet connector and set aside.
3. Disassemble mouthpiece or mask from cap. Open chamber by turning cap counterclockwise and remove baffle.
4. Wash all items, except tubing, in a hot water/dishwashing detergent solution. Rinse under hot tap water for 30 seconds to remove detergent residue.
5. Using a clean container or bowl, soak items in three parts hot water to one part white vinegar for 30 minutes **Figure 9** OR use a medical bacterial-germicidal disinfectant available through your provider. Be sure to follow manufacturer's instructions carefully.
6. With clean hands, remove items from disinfectant solution, rinse under hot tap water and air dry on a clean paper towel. Store in a zip-lock bag.
7. Keep the outer surface of the tubing dust-free by wiping regularly. Disposable nebulizer tubing does not

NOTE—Do not towel dry nebulizer parts; this could cause contamination.

CAUTION—Never place any nebulizer parts in an automatic dishwasher; doing so could cause damage.

WARNING—To prevent possible risk of infection from contaminated cleaning solutions, always prepare fresh solution for each cleaning cycle and discard solution after each use.

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have to be washed because only filtered air passes through it.

NOTE—The DeVilbiss 4650D-620 disposable nebulizers will last at least 15 days and possibly longer, depending on usage. Proper cleaning will also help extend the life of the nebulizer. Because it is disposable, we recommend that an extra nebulizer be kept on hand at all times. Sunrise also manufactures the Model 700 DeVilbiss reusable nebulizer that is dishwasher safe and may be cleaned and reused for up to one year.

Compressor Cleaning

1. With power switch in the "Off" position, unplug power cord from wall outlet.
2. Wipe outside of the compressor cabinet with a clean damp cloth every few days to keep dust-free.

Filter Change

CAUTION—Do not submerge in water; doing so will result in compressor damage.

1. Filter should be changed every 6 months or sooner if filter turns completely gray in color.
2. Remove filter cap by grasping it firmly and pulling out of the unit **Figure 10**.
3. Remove dirty filter with fingers and discard.
4. Replace with a new DeVilbiss filter (part #5650D-602). Additional filters should be purchased from your Sunrise provider.
5. Push filter cap with new filter into position.

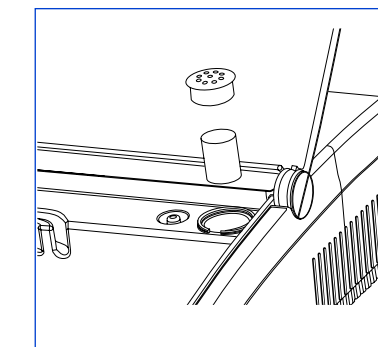
Figure 10/Figura 10

Maintenance

CAUTION—Reusing filter or substituting any other material such as cotton for a Sunrise air-inlet filter will result in compressor damage.

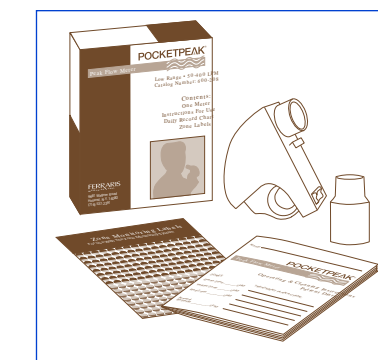
All maintenance must be performed by a qualified Sunrise provider or authorized service center.

Contact your Sunrise provider for information on our other products:

DANGER—Electric shock hazard. Do not remove compressor cabinet. All disassembly and maintenance must be done by a qualified Sunrise provider.

POCKETPEAK®

Your physician may also prescribe a peak flow meter to aid in early detection of airway restriction. This device shows how well you are breathing, and helps you to anticipate potential problems. Comparing your peak flow to a "normal" peak flow will help your physician determine the best therapy.

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